

Dist-inject

Pistol Model 30

Twisting the locking ring, to releases the barrel from the pistol. **(pic.1)**

Before inserting the syringe into the barrel, remove the oil or grease. For shooting, the barrel must be dry.

The prepared syringe is fully inserted into the barrel which is then replaced by aligning the foresight with the stop pin and secured by twisting the locking ring in the opposite direction. **(pic.2)**

Pressing the knob releases the breech catch and the barrel can be tilted downwards, exposing the cartridge chamber. **(pic.3)**

Insert the cartridge required in the chamber and snap the barrel shut. Make sure that the breech is fully closed by checking that there is no gap between the bolt and the cartridge chamber. **(pic.4)**

Immediately before firing, cock the pistol by pulling the hammer backwards until it engages. **(pic.5)**

Attention ! The safety catch is now off. (pic.6)



If for any reason, the pistol is not fired, grip the hammer firmly and ease it forward, at the same time pull the trigger.

The pistol is safe as long as the hammer is adjacent to the breech block.

Important ! To prevent the firing-pin from damaging the cartridge chamber, the trigger should only be pulled if there is a cartridge or a spent case in chamber.

After firing, the barrel and the cartridge chamber ought to be dry cleaned with the two rods supplied and then thoroughly oiled. An oil spray is recommended to keep the other metal parts in good condition. When storing the pistol, release its barrel and make sure that all superposed metal parts are sufficiently oiled.

Cartridges: Art Nr. 2012 „Blue“ up to 15 m.

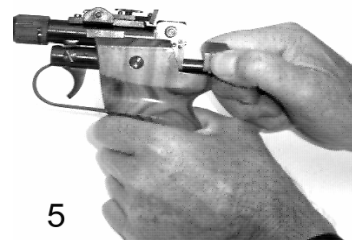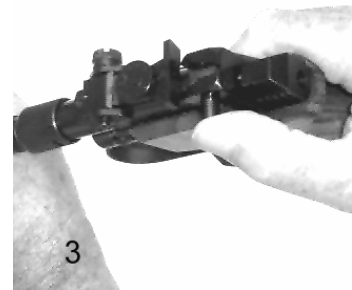

Dist-inject

Art. Nr. 2014 „Red“ from 12 to 25 m.