

## Dist-inject

### The Rifle Model 60

The breech is opened by pulling the breech catch in direction of the arrow. **no.1& 2**

All traces of grease should first be removed from the barrel. This operation is performed with the cleaning rod, the end being swathed with a piece of rag.

The bronze brush should only be used for dry cleaning preferably after each shot.

The sight is inserted in the slot and screwed tight by means of a coin.

The prepared syringe is inserted into the barrel, **no.3**

followed by cartridge holder. **no. 4 & 5** The O-ring at the cartridge holder has often to be greased with silicone in order to prevent a drop in pressure. The same operation is recommended for the last gasket of the syringe stabilizer.

After the insertion of the desired cartridge the rifle is vigorously closed and **put at safe by pulling back the safety knob. Release it only immediately before firing.**

By means of the shooting table put rifle sight leaf in desired position.

**Example:** Distance 40 m  
Cartridge Wight Art Nr. 2015  
Syringe 3 ml  
Position of sight leaf = 7



1



2




3



4



5

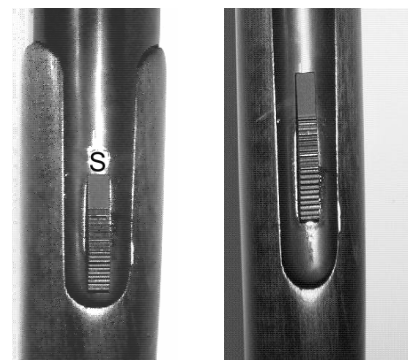

The cartridge holder with the eccentric cartridge opening has **always to be inserted at the same position.** Remark the design at its envelope.

After the shot, pull back the breech catch, open the breech and remove the cartridge holder. Eject spent cartridge from the holder by using the pin. Clean and grease the rifle after every use.

**Important !** Never operate the trigger when there is no cartridge or spent case in the cartridge holder.